

GLUCOCARD[®] EXPRESSION[™]

BLOOD GLUCOSE MONITORING SYSTEM

Competency Checklist

Operator _____

Date _____

Trainer _____



The Trainer should observe/confirm the Operator performs/understands the following:

1. Identifies test strip port location..... Yes No
2. Performs system setup (sets year, month, date, time, volume, and language) Yes No
3. Process of changing battery Yes No
4. Prepares lancing device correctly* Yes No
5. Dispose of lancet properly Yes No
6. Insert test strip into meter correctly Yes No
7. Apply blood sample-application tip correctly* Yes No
8. Identify when to repeat test..... Yes No
9. Performs and saves control solution test correctly Yes No
10. Confirms control solution test is in range Yes No
11. Proper steps in caring for the meter..... Yes No
12. Identifies common error codes Yes No
13. Reviews online video resources (meter set up, how to test your blood glucose, viewing meter memory)..... Yes No

Did the Operator perform each task correctly? Yes No

If yes, the Operator is now certified to use and train patients to use the GLUCOCARD[®] Expression[™] Blood Glucose Monitoring System. If no, perform tasks on list and document performance on a new competency checklist.

*Actual blood sampling is not required.