

# GLUCOCARD<sup>®</sup> Expression<sup>™</sup>

AUDIO-ENABLED BLOOD GLUCOSE MONITORING SYSTEM

## Performing a Control Solution Test

### When to Perform a Control Solution Test

- Using the meter for the first time.
- Using a new bottle of test strips.
- You left the test strip bottle cap open.
- You dropped the meter.
- You suspect your meter and test strips are not working properly.
- The blood glucose test results do not reflect how you feel.
- You want to practice the testing procedure.



For questions about your meter, call ARKRAY Technical Customer Service at 800.566.8558.  
Additional control solution can be requested by calling 800.818.8877.

For questions about your diabetes and medication management, call your health care provider.

\*Control solution range may vary. Step 8 range is example only.